

BeneHeart C & S Series AED Quick Reference Guide

1



Confirm that the patient is unresponsive with no breathing or not breathing normally.

CALL EMERGENCY MEDICAL SERVICE!


2



Open the AED lid.

3

Apply the pads.

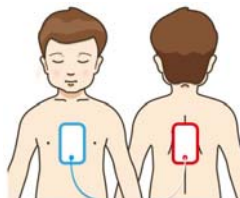

If Shock Advised

Fully-automatic model:
The AED automatically shocks the patient.

Semi-automatic model:
Press the Shock button within 30s.



4

Deliver a shock.

If No Shock Advised

Perform CPR according to the prompts.



046-017200-00 (2.0)

mindray